

# PARI **UNI** *light* mobil

Effective nebuliser therapy  
any time, anywhere!



## The PARI **UNI light mobil** nebuliser

- the mini nebuliser for all occasions
- fits in your handbag
- high percentage of fine, lung-accessible droplets (60% less than 5  $\mu\text{m}$ )
- high aerosol output for short inhalation times (380 mg/min)
- guaranteed proven PARI quality



Depending on your individual requirements, choose the version that suits you:

	<b>PARI UNI light</b> – basic compressor	<b>PARI UNI light mobil</b> – the complete package with battery pack
<b>Equipment:</b>	Compressor-driven inhalation device with PARI LC PLUS nebuliser and bag	Compressor-driven inhalation device with PARI LC PLUS nebuliser, rechargeable battery with up to 55 minutes inhalation time, 12 volt connection cable and bag
<b>Weight:</b>	approx. 400g	approx. 780 g
<b>Size:</b>	131 mm x 93 mm x 45 mm	131 mm x 93 mm x 74 mm
<b>Can be used:</b>	on all mains voltages	on all mains voltages, with car adapter and rechargeable battery for operation any time and anywhere
<b>Battery:</b>	N/A	LED displays remaining battery life. Capacity up to 55 minutes inhalation time. 255 charging cycles possible

# PARI UNI *light* mobil

## Unlimited mobility!

Small, compact, mobile. The PARI UNI light mobil is the reliable partner for inhalation therapy. Whether at home or on the road, at work or on vacation, it's always ready for use. Wherever you are. You can rely on PARI's reputation for quality.

### Indications:

Diseases of the respiratory tract, e.g. asthma, bronchitis, rhinitis, obstructive pulmonary diseases, changes in the respiratory tract (atelectasis or bronchioelectasis), cystic fibrosis.

Important hygiene benefits – all PARI nebulisers are suitable for:

- dishwashers
- sterilising in boiling water
- autoclaving
- chemical disinfection

### Aerosol characteristics:

TOR (total Output rate)	380 mg/min
MMD (median mass diameter)	4,2 µm
Percentage by mass	
% under 5 µm	60%
% under 2 µm	25%

Measurement with the Malvern Master Sizer X at 23° C and 50 % relative humidity. Nebulised medium: 0,9 % NaCl (5 ml). Inspiratory flow 20 l/min



# PARI UNI *light* mobil

## The two versions of the PARI UNI light


PARI **UNI light**  
with PARI LC PLUS® nebuliser,  
power cord and bag  
Order No. 046G1000



PARI **UNI light mobil**  
with PARI LC PLUS® nebuliser,  
rechargeable battery, 12 volt  
connection cable, power cord  
and bag  
Order No. 046G1100




To guarantee the effectiveness and quality of your nebuliser therapy,  
certain parts of the device should be regularly replaced:

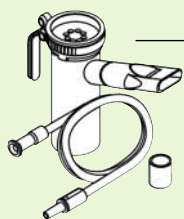

**PARI LC PLUS Year Pack**  
With PARI LC PLUS nebuliser,  
filter, tubing  
Order No. 022G8010

**Replacement  
rechargeable battery**  
Order No. 046G2000

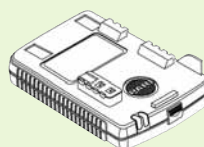

Satisfy yourself as to the quality of our products!  
For suggestions and questions:

PARI Export Service Center: 0049 (0) 8151 279-220  
www.pari.de · E-mail: info@pari.de

## PARI Export Service Center



0049 (0) 8151 279-220

Call...



... for questions on use, handling,  
cleaning or maintenance of your  
PARI nebuliser or inhalation device.



... for information about spare  
parts for today's and yesterday's  
PARI products and/or about  
where to order these.



... for additional information  
or individual questions about the  
entire PARI product range.

local distributor



PARI GmbH  
Spezialisten für effektive Inhalation  
Moosstrasse 3 · D-82319 Starnberg  
Telefon + 49 (0) 8151 279-0  
Fax + 49 (0) 8151 279-101



You can find PARI on the Internet:  
www.pari.de  
E-Mail: info@pari.de

Specialists in effective inhalation

